



Small Animal Exposure Chart for Computerised Radiography Systems – Dental

Suggested exposures are for a Focal Spot/Film Distance (FFD) of 80cm

Chart for conventional X-ray generator

	Suggested kV	Suggested mAs	Adjusted kV	Adjusted mAs
Cat				
Single tooth	50 – 55	3.2 – 4		
Occlusal	50 – 55	3.2 – 4		
Toy Dog				
Single tooth	50 – 55	3.2 – 4		
Occlusal	50 – 55	3.2 – 4		
Medium Dog				
Single tooth	55 – 60	4 – 6.4		
Occlusal	55 – 60	4 – 6.4		
Large Dog				
Single tooth	60 – 65	5 – 8		
Occlusal	60 – 65	5 – 8		

*** These charts are for taking small animal dental images with a conventional X-ray generator, for best image quality, and ease of positioning we recommend the use of a dental specific generator.**

Contact us now

www.imv-imaging.co.uk
info@imv-imaging.com
 +44 (0) 1506 460023
 Follow us