



Step 1

- ① Tap 'New...' to get started
- ② Detector is ...

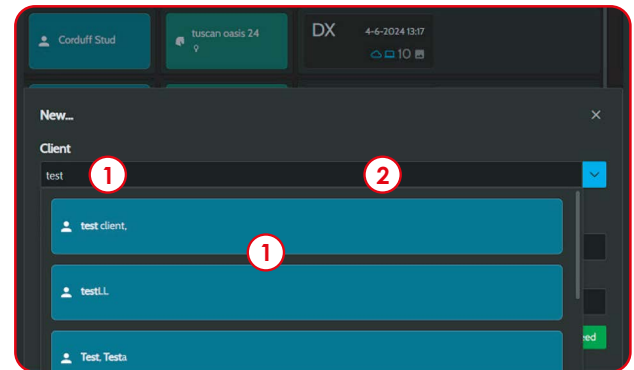


Step 2

- ① Search for an existing client and select from the list **or**
- ② Fill in the name of a new client

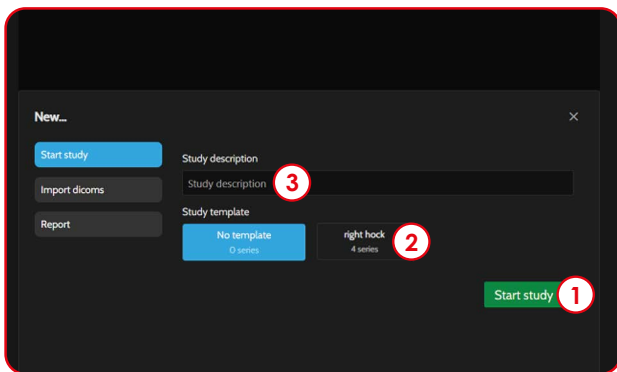


Step 3

- ① Tap 'Start study' to proceed **or**
- ② Select a template then tap 'Start study'
- ③ *Optional:* enter a description for your study

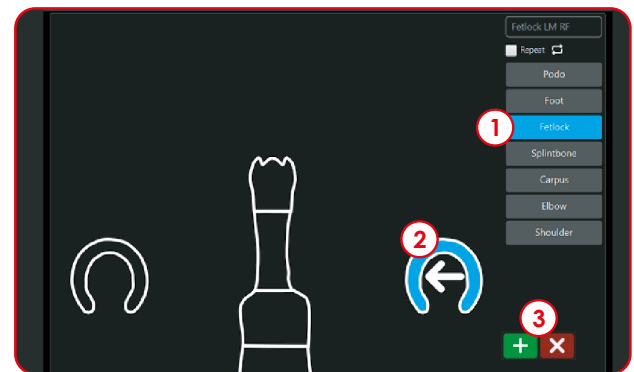

Step 4

- ① Select a view and click '+' to add it (or double-tap it)
- ② Swipe or tap for the direction
- ③ '+' = add selected view, 'x' = close window

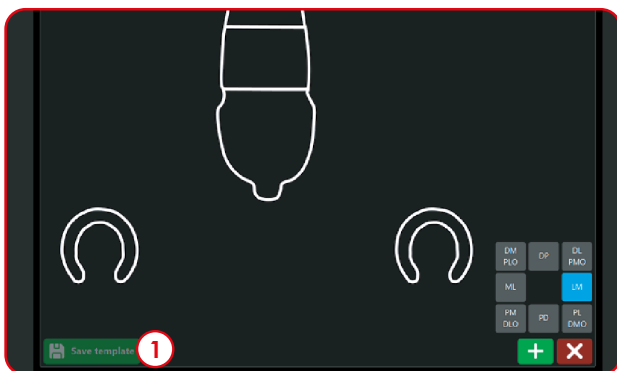


Step 5

- ① Tap 'Save template' if you will use this set often



Step 6


- ① If the detector detects x-ray exposure, the image will be displayed
- ② The next view is selected

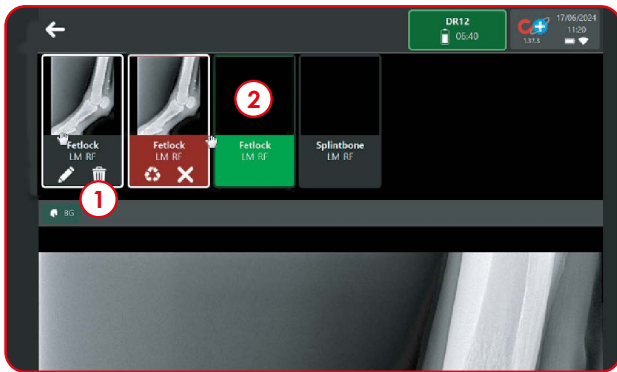

Step 7


- ① Tap the bin to take the image again
- ② A new view is added automatically

Deactivated images (red) are saved, but not shared

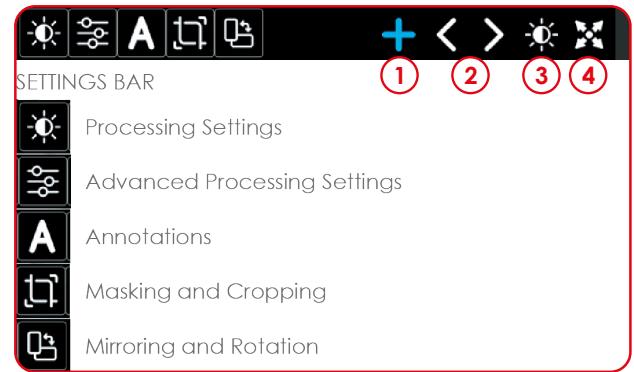


Step 8

- ① Tap '+' to return to the selector (for views)
- ② Select the previous or next image
- ③ Reset brightness/contrast
- ④ Reset zoom level

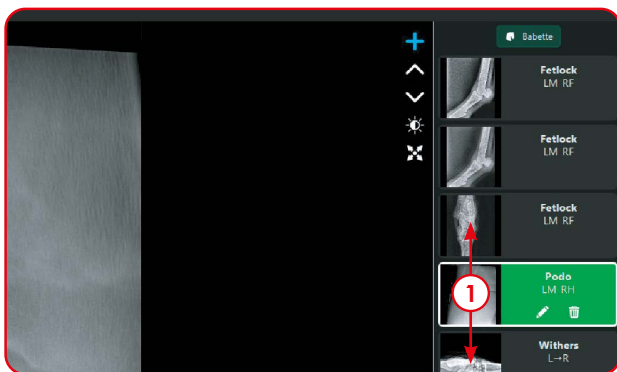


Step 9

- ① Grab (touch and hold) a thumbnail and slide it up or down to change its place in the list (re-order)

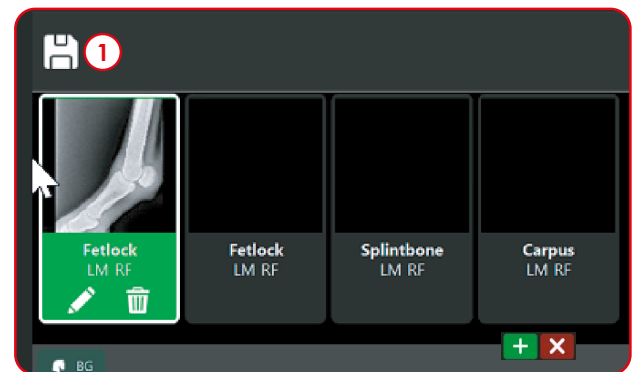

Step 10

- ① Tap the Disk icon to save the study and exit the acquisition

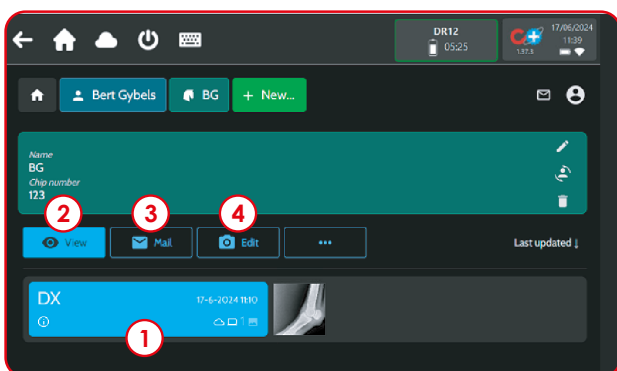

Step 11

- ① Select a study and tap ② to open it in the viewer, ③ to mail it to your client, or ④ to return to acquisition and edit the study

Contact us now

UK +44 (0) 1506 460 023
IE +353 (0) 42 932 0070

support@imv-imaging.com

veterinarysolutions.be

Follow us 

Tablet
quick
guide

